


# Exporting from PreXion

1. On your desktop, right click on an empty space. Go to “New” and click “Folder”. A folder will appear with highlighted blue text. Type in the patient name and push “Enter” on your keyboard.

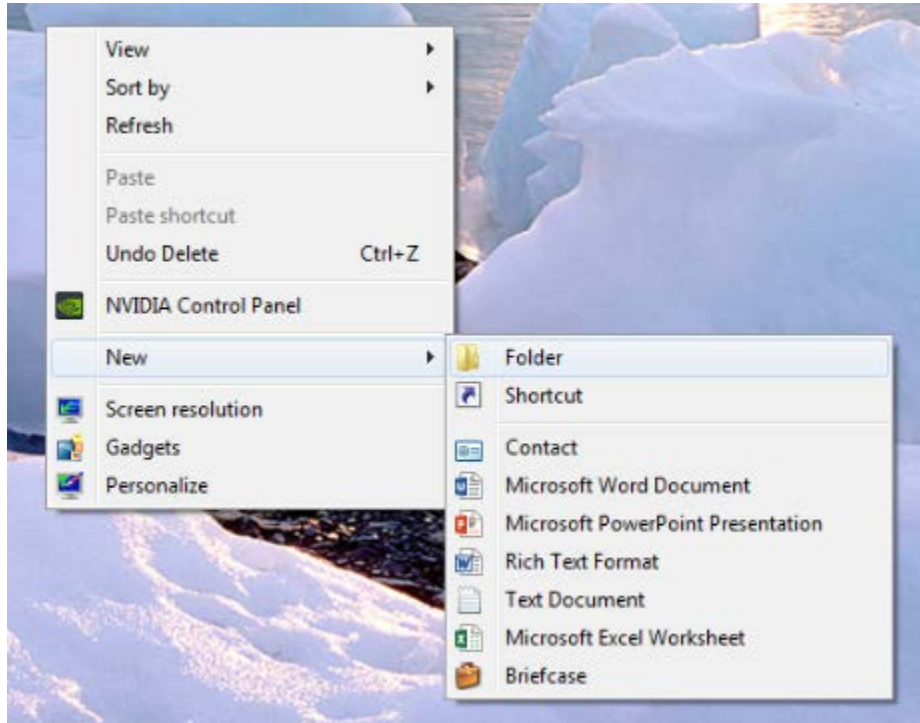

2. In your PreXion, right click on the patient’s name and go to “Study Management” and click “Download Study”.

PreXion 3D [Monday, October 17, 2016: 10:13]


Patient Information

#	Patient ID	Patient	Modality	Study Date, Time	Age	Gender	Description	AE Title	DOB	Study ID	Accession#
1	01450		CT	2016/10/13 16:00:34	53	F	np		1963/07/14		
2	01512	Patient Name	CT	2016/06/20 15:27:50	62	F	impl				
3	01450		CT	2016/06/20 14:22:30	52	F	impl			pano	
4	01450		CT	2016/05/12 15:10:13	52	F	impl				
5	0110		CT	2016/04/11 15:31:27	62	F	np				


Context menu options:

- Quick Download
- Get list of series with this PatientID
- Add To Multi-Data List
- Study Management
  - Anonymize Study
  - Download Study
  - Push Study
  - Retrieve Study
  - Delete Study
  - Save Study Info to File

3. Select the folder you just created on the desktop. Click “OK” to export the images.

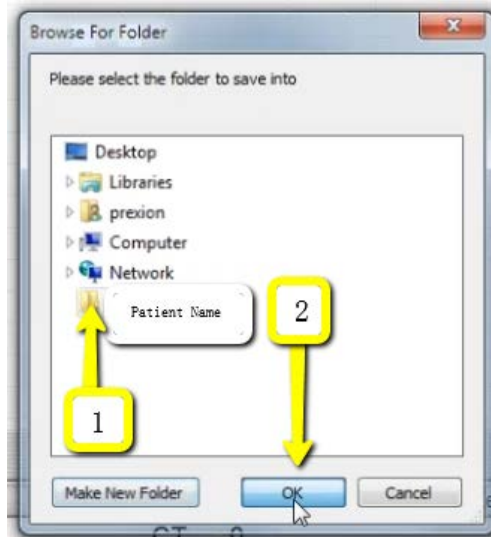


4. After the DICOM exports and saves, go to the patients folder and verify the .dcm files were exported correctly; there should be several hundred files.

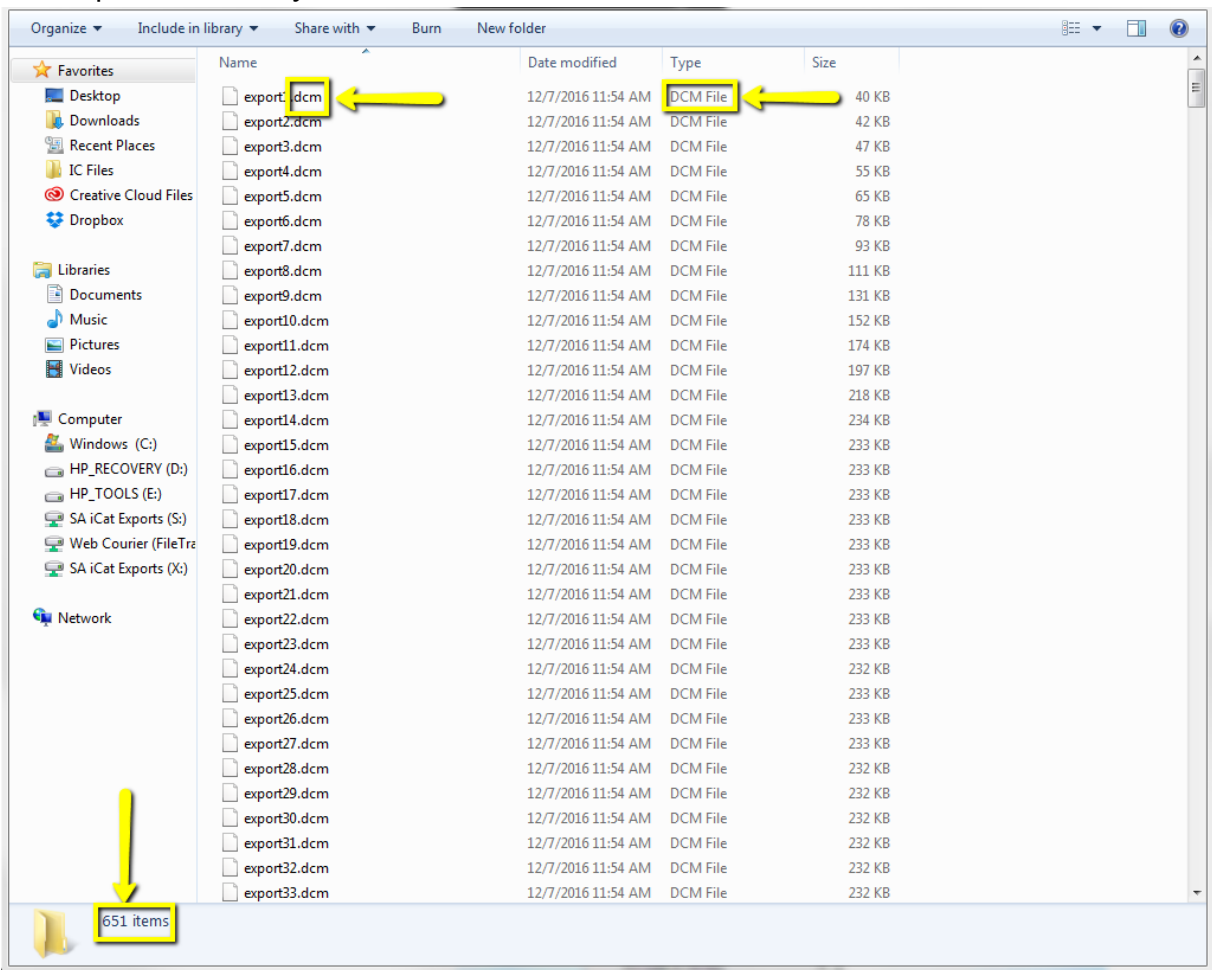


5. Go back to the desktop, right click on the patients folder, go to “Send to”, and select “Compressed (zipped) folder”. A duplicate folder will be made with either a zipper on it, or a blue “Z”.

(\*\*\* If following a Dual Scan Protocol, please zip scans individually and label accordingly; i.e: “Patient Scan” and “Denture Scan” \*\*\*)

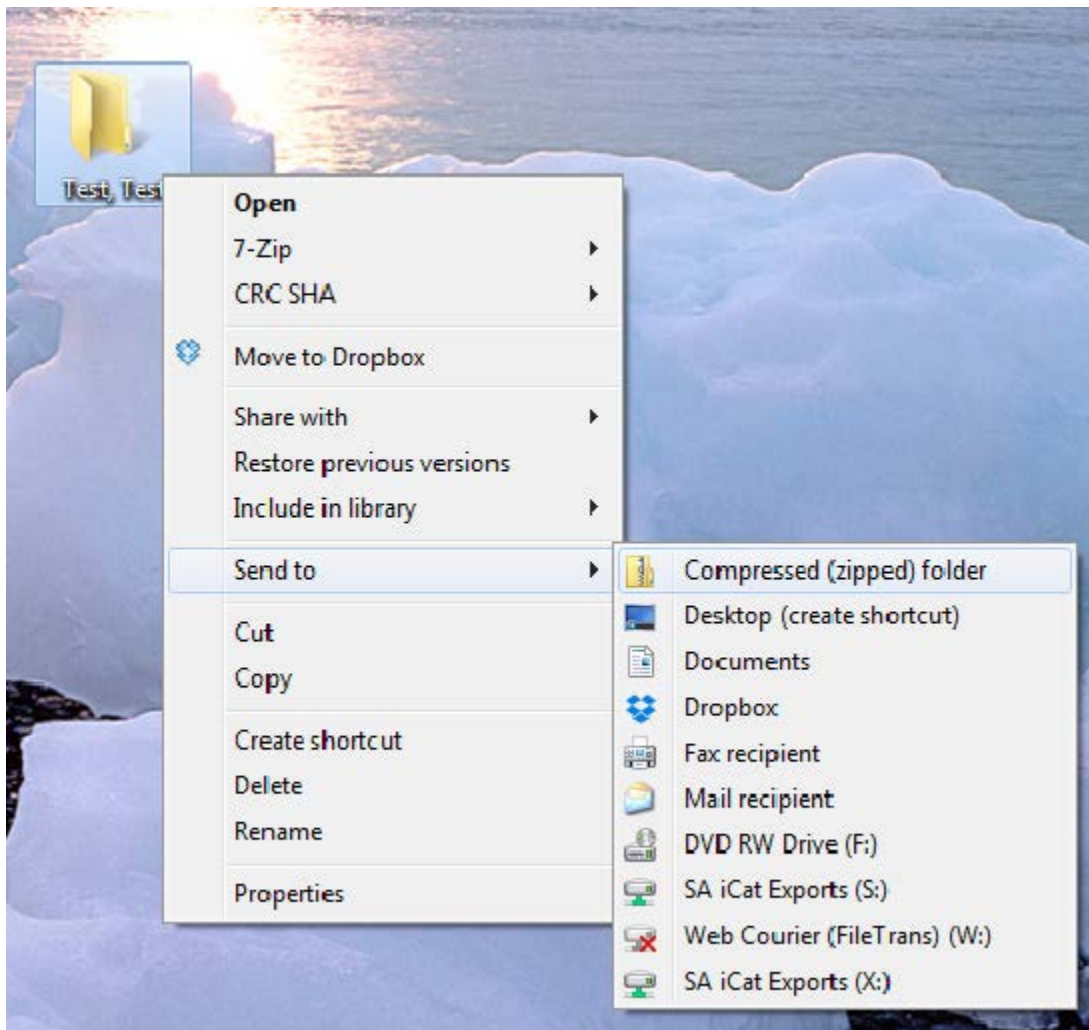

6. To upload the compressed folder, login to Implant Concierge. Click on the patient's name to open the case. Click on the red "Upload DICOM" button.

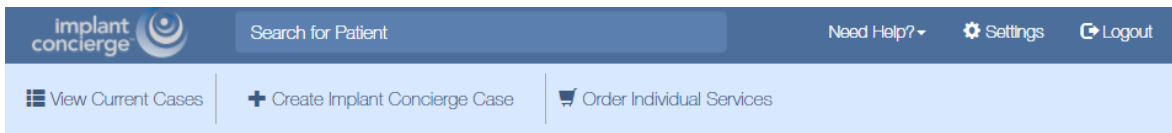


implant concierge Search for Patient Need Help? Settings Logout

View Current Cases Create Implant Concierge Case Order Individual Services

## Current Cases

In Progress Pending Partner Completed All Cases Invoice(s)

⊙ Indicates an action that has been delayed.

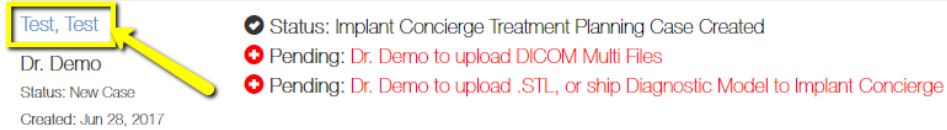

Test, Test  
Dr. Demo  
Status: New Case  
Created: Jun 28, 2017

- Status: Implant Concierge Treatment Planning Case Created
- ⊙ Pending: Dr. Demo to upload DICOM Multi Files
- ⊙ Pending: Dr. Demo to upload .STL, or ship Diagnostic Model to Implant Concierge




implant concierge Search for Patient Need Help? Settings Logout

View Current Cases Create Implant Concierge Case Order Individual Services

Case #1380740920 has been created and is waiting for its first action to be completed.  
Need help getting started? Call Customer Support: 866-977-2228

Dr. Test Demo

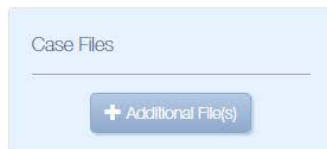

Case Files

+ Additional File(s)

### Test Test

Gender: Male | DOB: 0000

Options Case ID: 1380740920

Next Step

Dr. Demo to upload DICOM Multi Files



Upload DICOM Files

7. Drag and drop the folder from the desktop to Implant Concierge. The file will automatically begin uploading.



Upload DICOM Files

Patient: Test Test  
Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files  
or use the Add DICOM File button below


Once all files have been uploaded click Complete

+ Add DICOM File

Cancel Complete

## Upload DICOM Files

Patient: Test Test  
Case Number: 1380740920

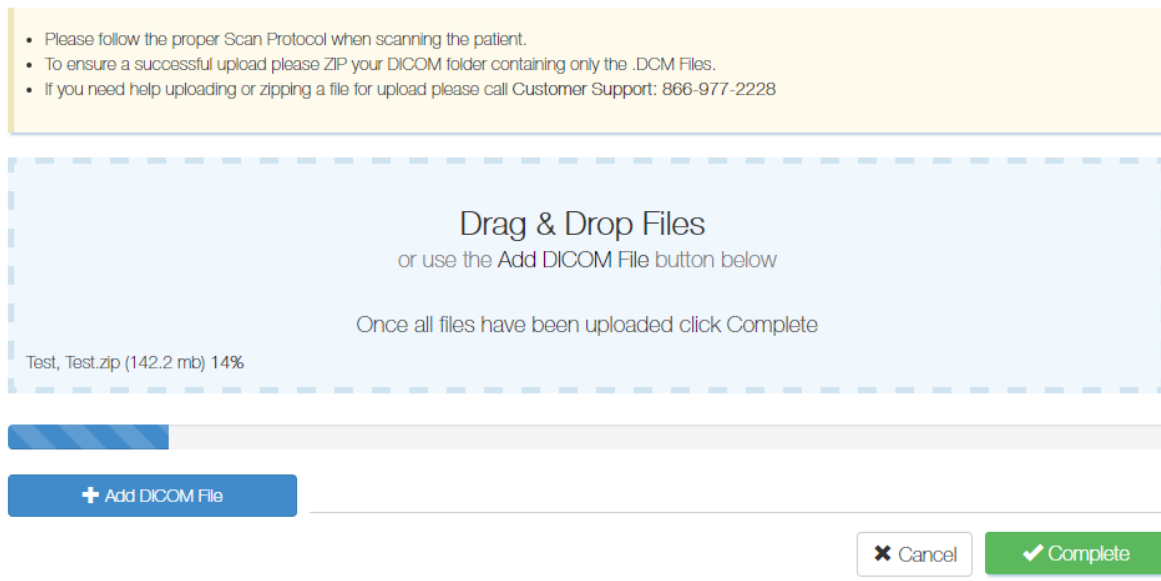

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files  
or use the Add DICOM File button below

Once all files have been uploaded click Complete

Test, Test.zip (142.2 mb) 14%

+ Add DICOM File

Cancel Complete

8. Once the file reaches 100%, click the green “Complete” button.

## Upload DICOM Files

Patient: Test Test

Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

### Drag & Drop Files

or use the Add DICOM File button below



Test, Test.zip (142.2 mb) 100%

Once all files have been uploaded click Complete

+ Add DICOM File

✕ Cancel

✓ Complete