



Exporting from NewTom



1. On your desktop, right click on an empty space. Go to “New” and click “Folder”. A folder will appear with highlighted blue text. Type in the patient name and push “Enter” on your keyboard.

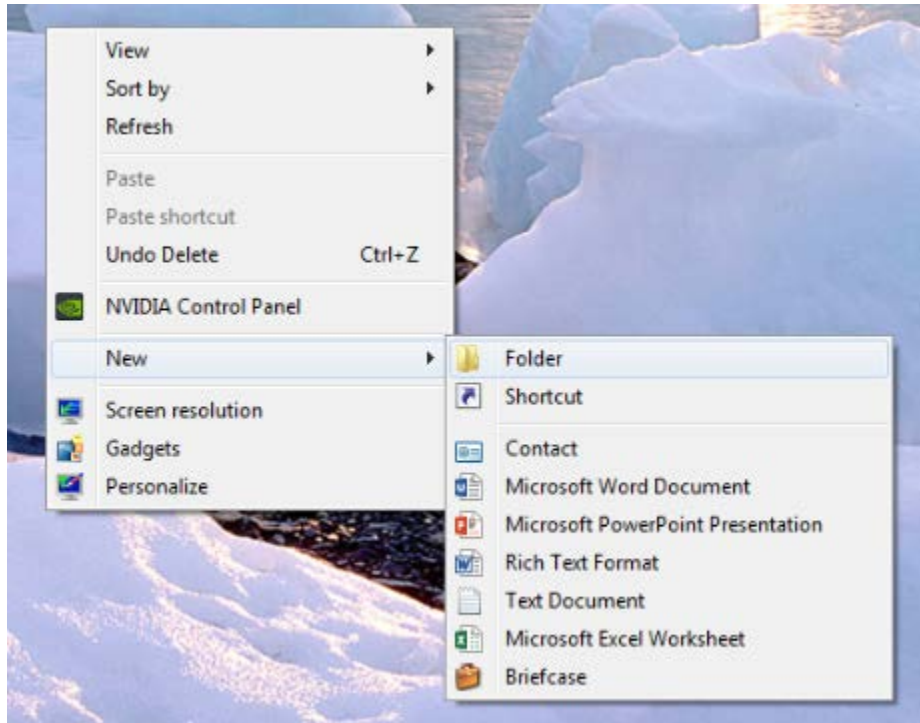


2. Open the patients scan. Select “File”, “Save in DICOM format”, “Sequence of Axial Images”, and select “Free Matrix”.

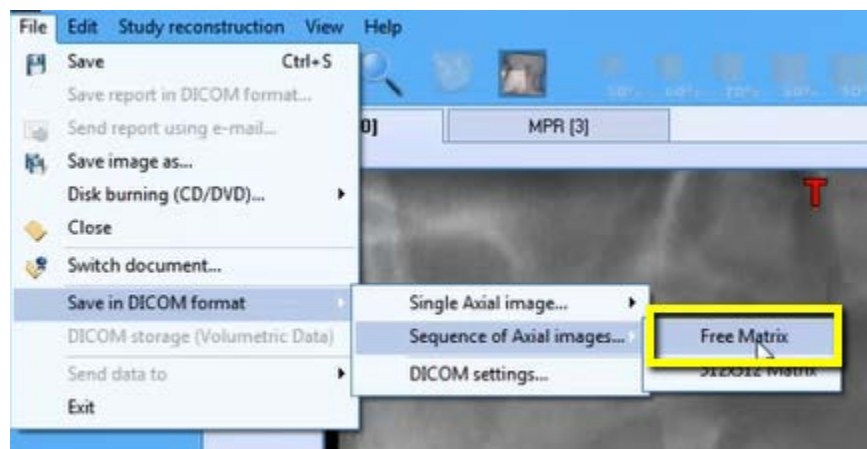

3. A pop-up will appear for the export location. Select the folder you just created on your desktop. Click “OK” to export images. Another pop-up will appear once DICOM has exported. Click “Ok”.

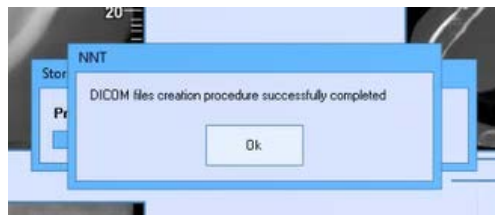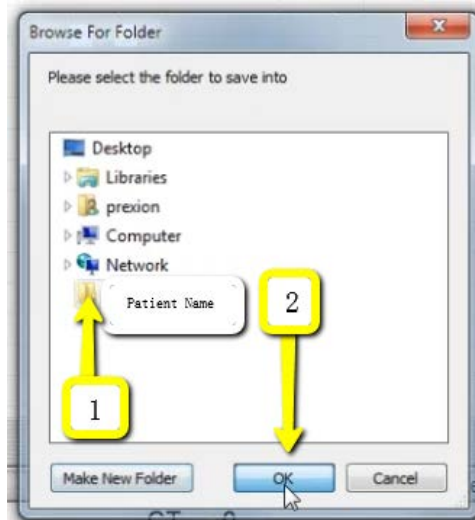


4. Go to the patients folder on the desktop and double click to open it. Verify the .dcm files were exported correctly; there should be several hundred files.

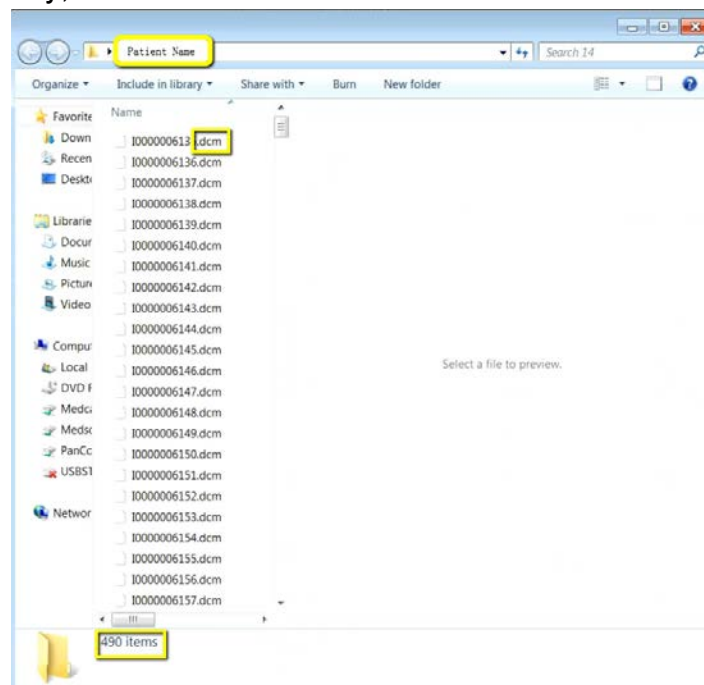


5. Go to the desktop, right click on the patients folder, go to “Send to”, and select “Compressed (zipped) folder”. A duplicate folder will be made with either a zipper on it, or a blue “Z”.

(*** If following a Dual Scan Protocol, please zip scans individually and label accordingly; i.e: “Patient Scan” and “Denture Scan” ***)

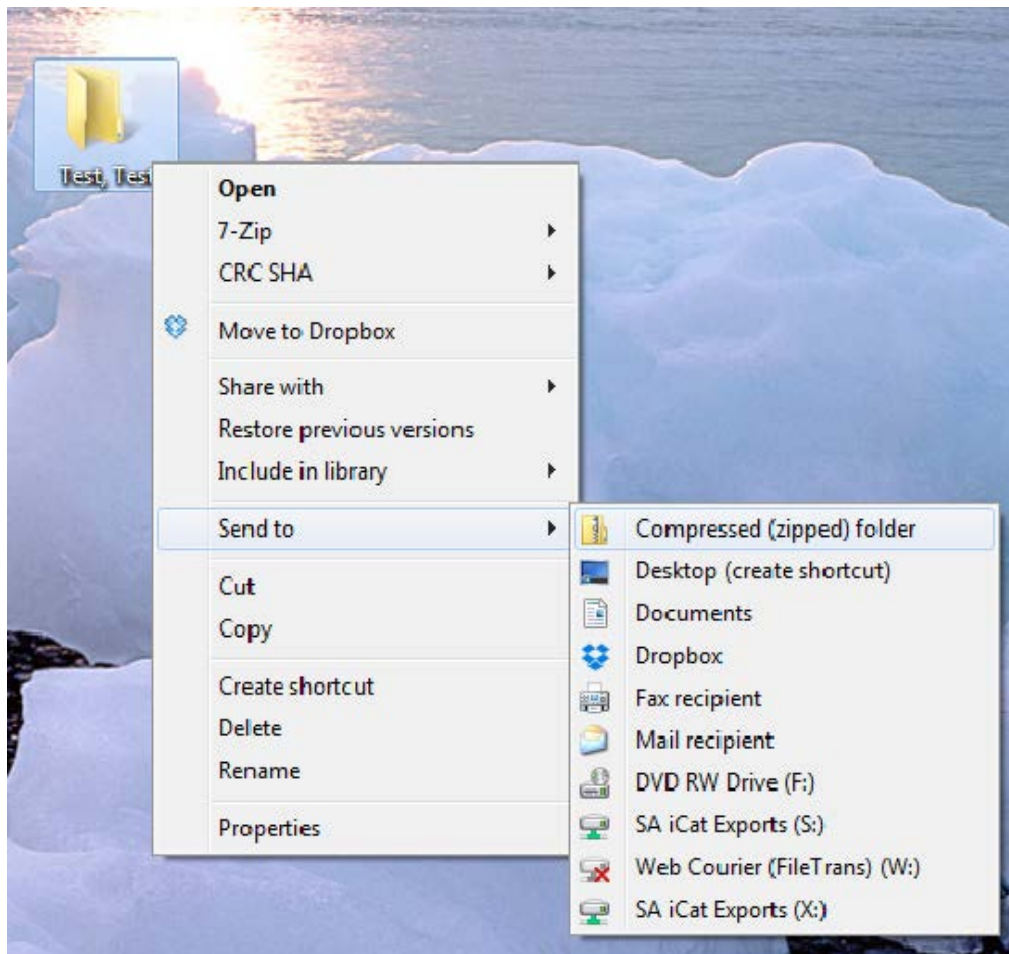

6. To upload the compressed folder, login to Implant Concierge. Click on the patient's name to open the case. Click on the red "Upload DICOM" button.

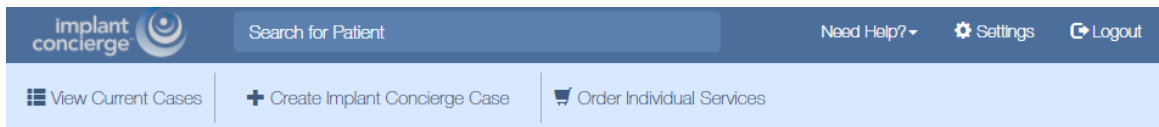

implant concierge Search for Patient Need Help? Settings Logout

View Current Cases Create Implant Concierge Case Order Individual Services

Current Cases

In Progress Pending Partner Completed All Cases

Invoice(s)

⊘ Indicates an action that has been delayed.

Test, Test

Dr. Demo

Status: New Case

Created: Jun 28, 2017

✔ Status: Implant Concierge Treatment Planning Case Created

⊘ Pending: Dr. Demo to upload DICOM Multi Files

⊘ Pending: Dr. Demo to upload .STL, or ship Diagnostic Model to Implant Concierge



implant concierge Search for Patient Need Help? Settings Logout

View Current Cases Create Implant Concierge Case Order Individual Services

Case #1380740920 has been created and is waiting for its first action to be completed.

Need help getting started? Call Customer Support: 866-977-2228

Dr. Test Demo

Test Test

Options

Gender: Male | DOB: 0000

Case ID: 1380740920

Case Files


+ Additional File(s)

Next Step

Dr. Demo to upload DICOM Multi Files

Upload DICOM Files

7. Drag and drop the folder from the desktop to Implant Concierge. The file will automatically begin uploading.

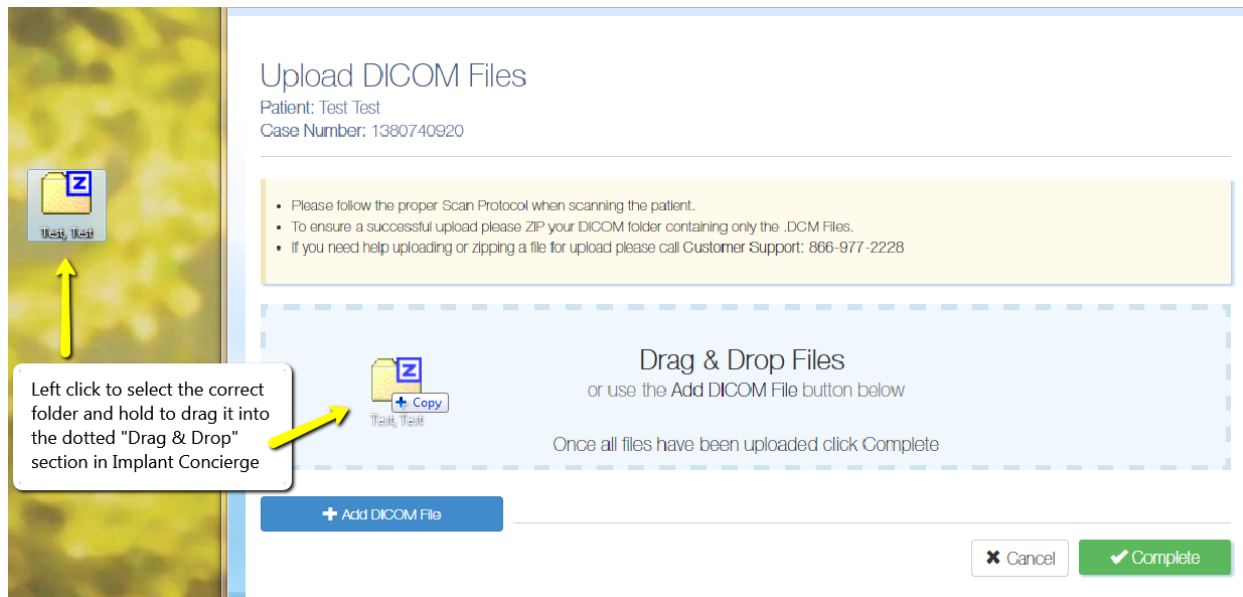

Upload DICOM Files
Patient: Test Test
Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files
or use the Add DICOM File button below

Once all files have been uploaded click Complete

+ Add DICOM File

Cancel Complete

Left click to select the correct folder and hold to drag it into the dotted "Drag & Drop" section in Implant Concierge

Upload DICOM Files

Patient: Test Test
Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files

or use the Add DICOM File button below

Once all files have been uploaded click Complete

Test, Test.zip (142.2 mb) 14%

+ Add DICOM File

Cancel

Complete

8. Once the file reaches 100%, click the green “Complete” button.

Upload DICOM Files

Patient: Test Test

Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files

or use the Add DICOM File button below

Once all files have been uploaded click Complete



Test, Test.zip (142.2 mb) 100%

+ Add DICOM File

✕ Cancel

✓ Complete