



# Export Blue Sky Bio Treatment Plan

In addition to the Blue Sky Bio treatment plan, Implant Concierge also requires the raw DICOM files, as well as a diagnostic cast (a digital impression or a stone model). More information can be found at the bottom of the page.


1. Once you have completed the treatment plan in your Blue Sky software, click “File” and “Save Project As...”

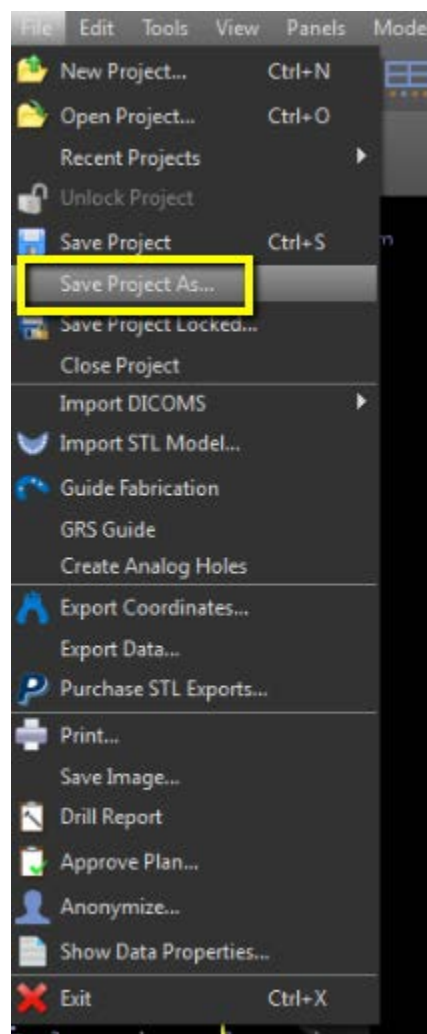


2. Select the Desktop and change the file name to “PatientName\_Treatment\_Plan.bsb”.  
Click “OK”

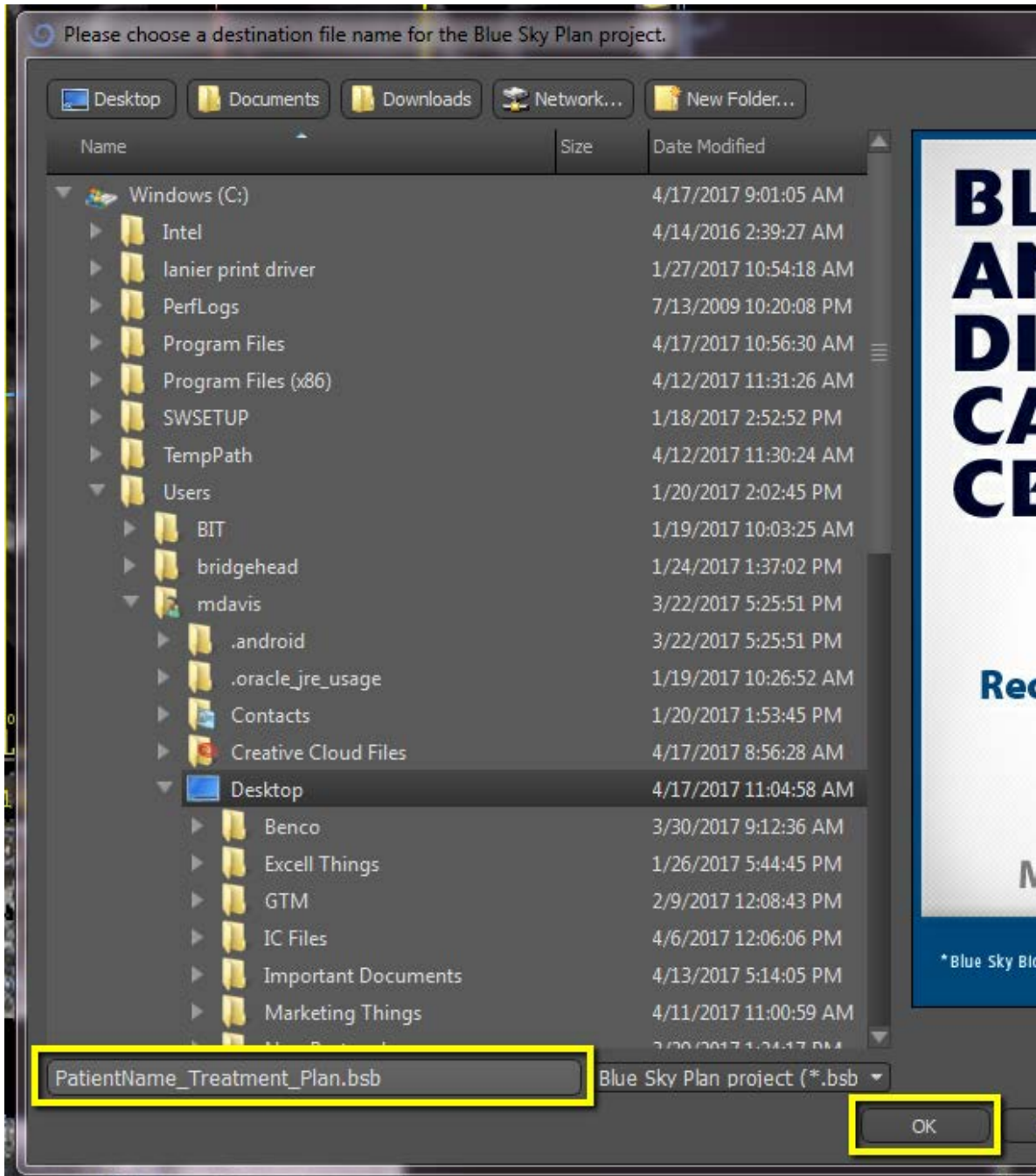

3. After the treatment plan saves, go to the Desktop, right click on the treatment plan, go to “Send to”, and select “Compressed (zipped) folder”. A folder will be made with either a zipper on it, or a blue “Z”.

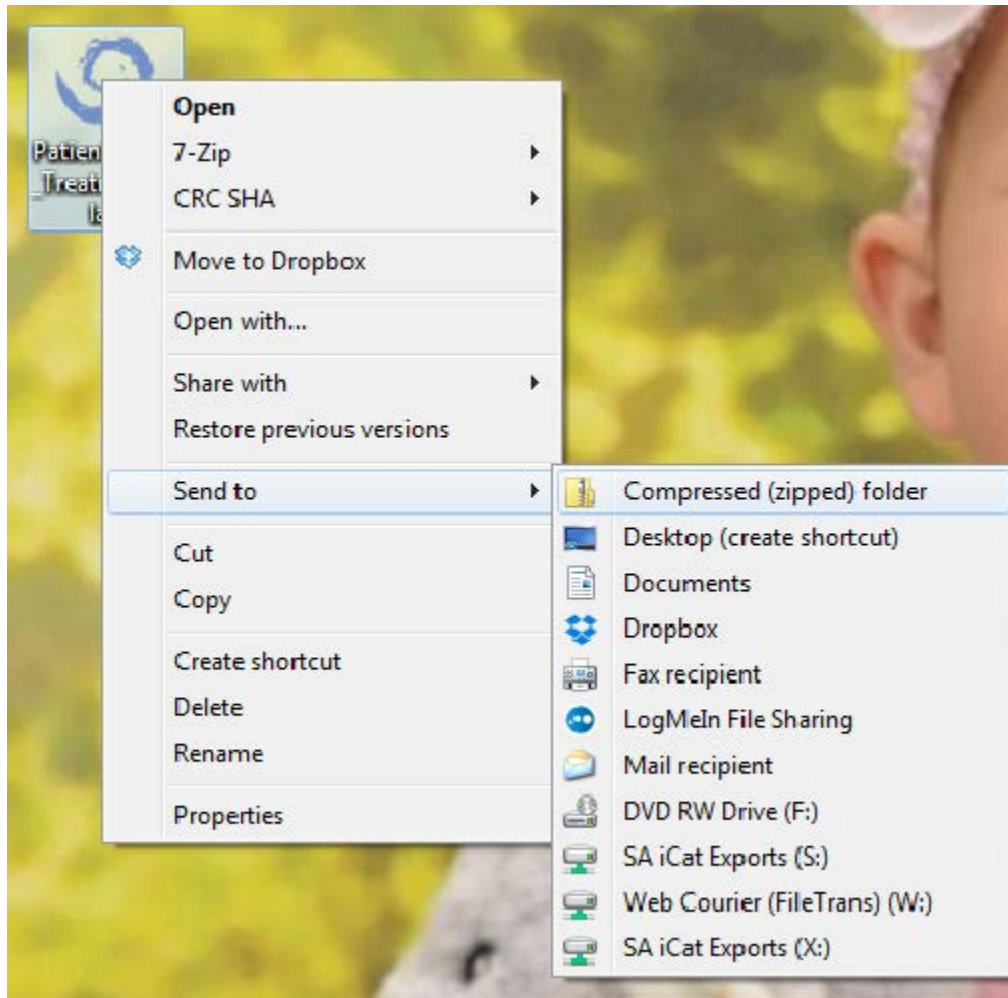

4. To upload the compressed folder, login to Implant Concierge. Click on the patient's name to open the case. Click on the red "Upload DICOM" button.

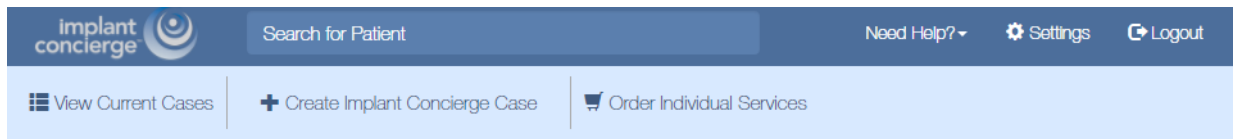

implant concierge Search for Patient Need Help? Settings Logout

View Current Cases Create Implant Concierge Case Order Individual Services

## Current Cases

In Progress Pending Partner Completed All Cases Invoice(s)

⌚ Indicates an action that has been delayed.

Test, Test	✔ Status: Implant Concierge Treatment Planning Case Created
Dr. Demo	⌚ Pending: Dr. Demo to upload DICOM Multi Files
Status: New Case	⌚ Pending: Dr. Demo to upload .STL, or ship Diagnostic Model to Implant Concierge
Created: Jun 28, 2017	



implant concierge Search for Patient Need Help? Settings Logout

View Current Cases Create Implant Concierge Case Order Individual Services

Case #1380740920 has been created and is waiting for its first action to be completed.  
Need help getting started? Call Customer Support: 866-977-2228

Dr. Test Demo

Case Files

+ Additional File(s)

### Test Test

Gender: Male | DOB: 0000


Options Case ID: 1380740920

Next Step

Dr. Demo to upload DICOM Multi Files

Upload DICOM Files

5. Drag and drop the folder from the desktop to Implant Concierge. The file will automatically begin uploading.

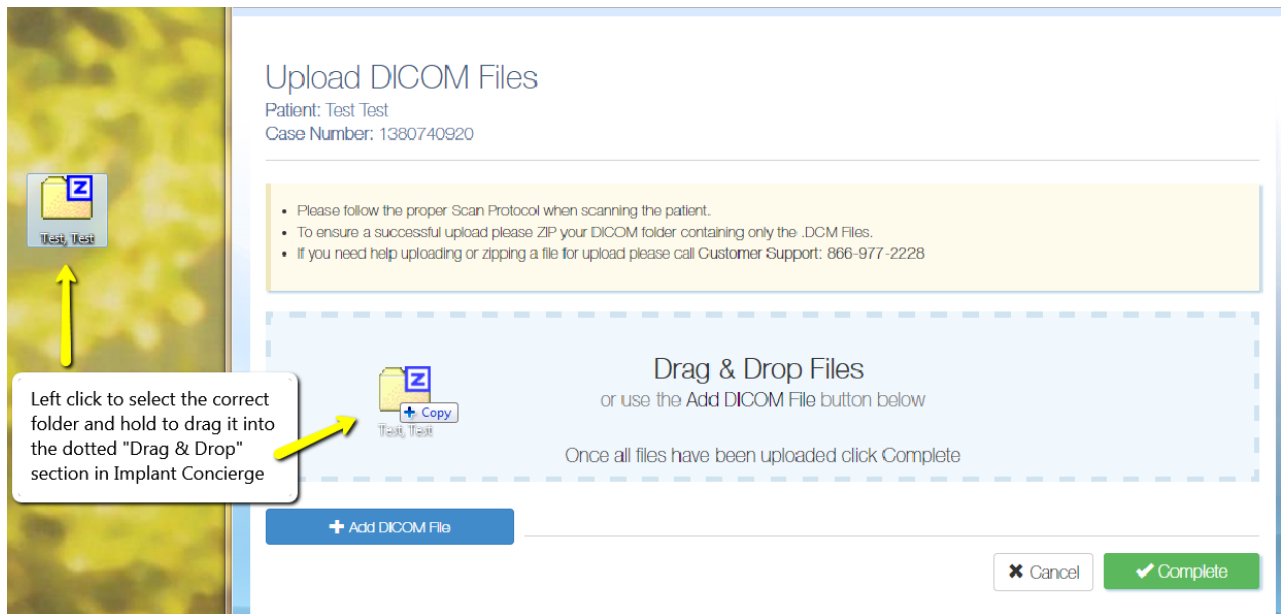

## Upload DICOM Files

Patient: Test Test  
Case Number: 1380740920


- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

### Drag & Drop Files

or use the Add DICOM File button below

Once all files have been uploaded click Complete

Test, Test.zip (142.2 mb) 14%

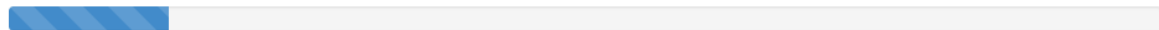

+ Add DICOM File

Cancel

Complete

6. Once the file reaches 100%, click the green “Complete” button.

## Upload DICOM Files

Patient: Test Test

Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

**Drag & Drop Files**  
or use the Add DICOM File button below

Once all files have been uploaded click Complete

Test, Test.zip (142.2 mb) 100%

+ Add DICOM File

✕ Cancel **✔ Complete**

For more information regarding Exporting DICOM and sending diagnostic casts, please use the links below:

[Exporting DICOM](#)

[How To Print a Shipping Label](#)