


Export Acteon Treatment Plan

When exporting your Acteon treatment plan, this will also include the multi-file DICOM. In addition to the treatment plan and multi-file DICOM, Implant Concierge will also require a diagnostic cast (a digital impression or a stone model).

1. In your Acteon software, left click the CBCT tile to highlight it in blue, then right click and select “Export Case”.



2. A pop-up will appear for the save location. Select “Desktop” and click “Select Folder”. This will export the DICOM and the viewer containing the treatment plan.

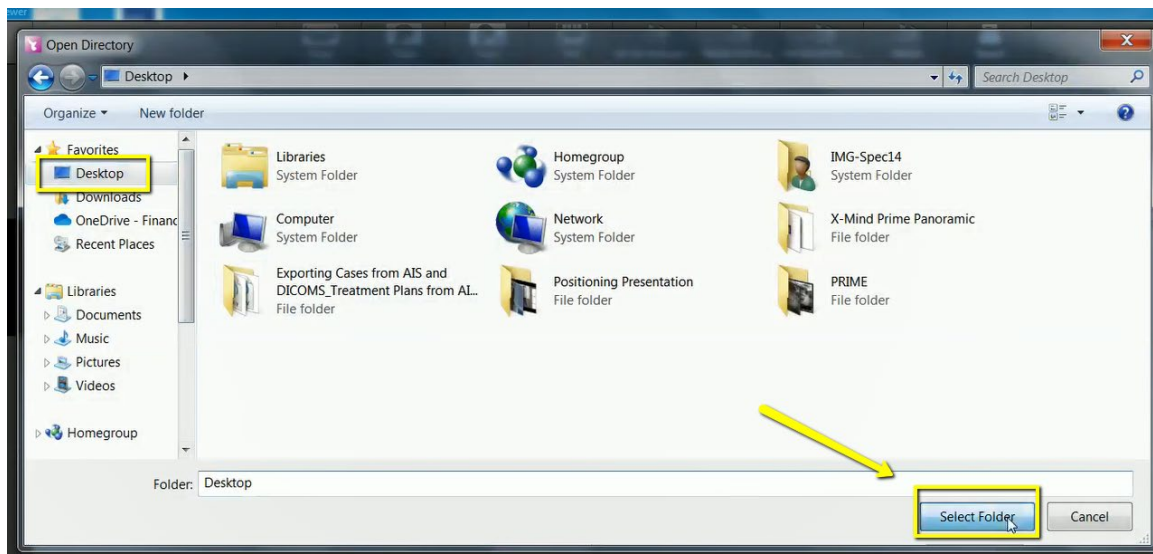





3. Once the Export is complete, open the folder to confirm the files saved properly. The raw DICOM files can be found under “Bin” > “Storage” > “PatientsName” > “CT”.










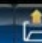


Name	Date modified	Type	Size
bin	6/12/2019 9:13 AM	File folder	
language	6/12/2019 9:13 AM	File folder	
Resources	6/12/2019 9:13 AM	File folder	
WFI4.0manuals	6/12/2019 9:13 AM	File folder	
start	6/19/2013 3:41 PM	Windows Batch File	1 KB

Name	Date modified	Type
3D_Templates	6/12/2019 9:13 AM	File folder
Storage	6/12/2019 9:13 AM	File folder
77a	2/3/2009 4:21 AM	Application


Name	Date modified	Type
 CT__15500_1474465930_7572 	6/12/2019 9:13 AM	File folder

The treatment plan can be found in the viewer by double-clicking on “start”. In the Acteon viewer, you will see a section containing actions for the treatment plan.


Name	Date modified	Type
 bin	6/12/2019 9:13 AM	File folder
 language	6/12/2019 9:13 AM	File folder
 Resources	6/12/2019 9:13 AM	File folder
 WFI4.0manuals	6/12/2019 9:13 AM	File folder
 start 	6/19/2013 3:41 PM	Windows Batch File

Treatment Plan (DICOM Included):			
Actions	Project name	Last modified	Description
   	Project1_2019-05-09_10-58-31	05/09/2019 (10:58:30)	

4. On the desktop, right-click on the patient’s folder, go to “Send to”, and select “Compressed (zipped) folder”. A duplicate folder will be made with either a zipper on it, or a blue “Z”.

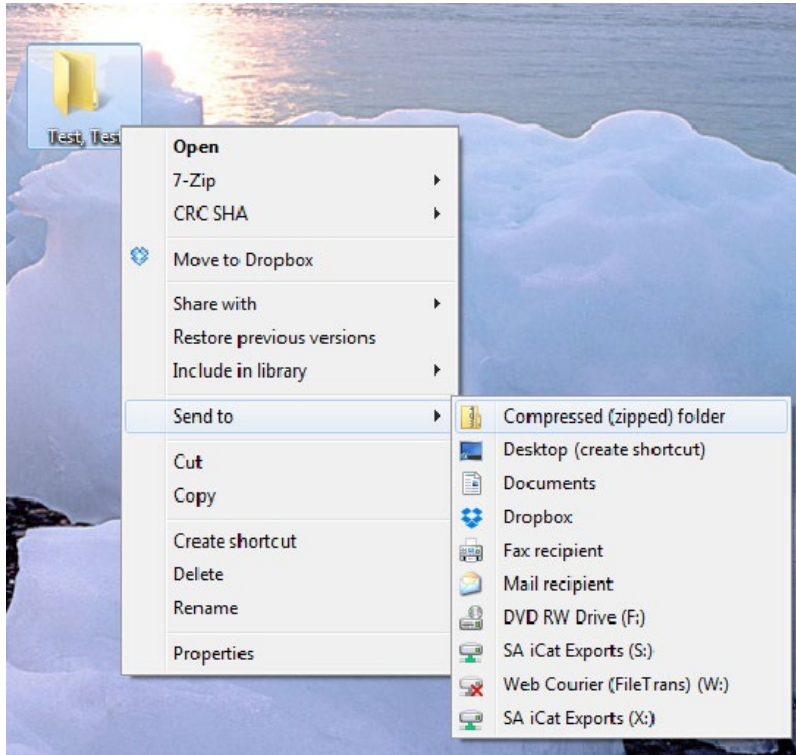

5. To upload the compressed folder, login to Implant Concierge. Click on the patient’s name to open the case. Click on the red “Upload DICOM” button.

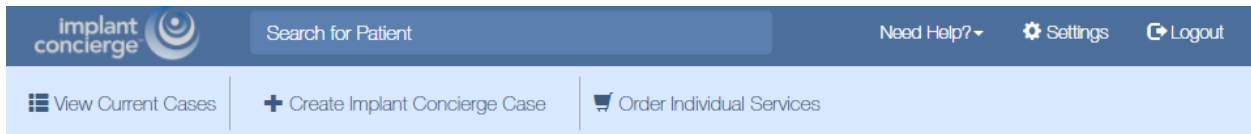

Current Cases

[In Progress](#)
[Pending](#)
[Partner](#)
[Completed](#)
[All Cases](#)
[Invoice\(s\)](#)

⌚ Indicates an action that has been delayed.

<div style="border: 2px solid yellow; padding: 2px; display: inline-block;">Test, Test</div> Dr. Demo Status: New Case Created: Jun 28, 2017	<ul style="list-style-type: none"> ✔ Status: Implant Concierge Treatment Planning Case Created ⊕ Pending: Dr. Demo to upload DICOM Multi Files ⊕ Pending: Dr. Demo to upload .STL, or ship Diagnostic Model to Implant Concierge
---	---

Case #1380740920 has been created and is waiting for its first action to be completed.
Need help getting started? Call Customer Support: 866-977-2228

Dr. Test Demo

Test Test

Options

Gender: Male | DOB: 0000

Case ID: 1380740920

Case Files

+ Additional File(s)

Next Step

Dr. Demo to upload DICOM Multi Files

Upload DICOM Files

6. Drag and drop the folder from the desktop to Implant Concierge. The file will automatically begin uploading.

Upload DICOM Files
Patient: Test Test
Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files
or use the Add DICOM File button below

Once all files have been uploaded click Complete

+ Add DICOM File

Cancel Complete

Left click to select the correct folder and hold to drag it into the dotted "Drag & Drop" section in Implant Concierge

Upload DICOM Files

Patient: Test Test

Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files

or use the Add DICOM File button below

Once all files have been uploaded click Complete

Test, Test.zip (142.2 mb) 14%

+ Add DICOM File

✕ Cancel

✓ Complete

7. Once the file reaches 100%, click the green “Complete” button.

Upload DICOM Files

Patient: Test Test

Case Number: 1380740920

- Please follow the proper Scan Protocol when scanning the patient.
- To ensure a successful upload please ZIP your DICOM folder containing only the .DCM Files.
- If you need help uploading or zipping a file for upload please call Customer Support: 866-977-2228

Drag & Drop Files

or use the Add DICOM File button below

Once all files have been uploaded click Complete

Test, Test.zip (142.2 mb) 100%

+ Add DICOM File

✕ Cancel

✓ Complete